Bath Sponge

VM971

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Bath Sponge from Aidapt. The Bath Sponge is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

This soft sponge is designed to help reach and clean your back. It features a 575 mm (20") angled plastic handle and hanging loop.

INSTRUCTIONS FOR USE

- The Bath Sponge can be used to reach and clean your back and hard to reach areas, this is particularly useful for those with limited mobility due to the long handle.
- Can be used in the bath or shower.
- Apply your favourite bath or shower gel to the dampened sponge and use as you would an ordinary bath sponge.
- Allow to air dry when not in use, can be stored using the integrated hanging loop.

CLEANING

Clean your Bath Sponge using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Allow the bath sponge to air dry in a warm dry spot after cleaning, away from direct heat sources.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)