

HKC01



Haika HKC01 Surgical Gown

Reusable Surgical Gown

Key Features

Ergonomic fit for freedom of movement

Secure back neck studded adjustment

Elasticated soft touch ribbed cuffs

Thumb loops for added security

One size fits all wraparound design for enhanced coverage

Easily adjusted with side tie fastening

Air permeable for added comfort

Machine washable up to 73°C for 100 washes

Quick drying



EN13795-1



Made in the UK

EU Compliance

- Class 1 Medical Device
- EN13795-1 Standard Performance

Ordering Information

- **Product Code:** HKC00001DZ
- **Pack Quantity:** 1 single per bag
- **Case Quantity:** 10 singles per carton

Haika HKC01 Surgical Gown

STORAGE AND MAINTENANCE

WASH INSTRUCTIONS

- Non-ionic detergent should be used
- Do not use bleach
- Do not iron
- Do not dry clean
- Wash at a maximum temperature of 73°C
- Tumble Dry exhaust temperature should not exceed 60°C. Controlled cooling, after wash, to 40°C, will minimise creasing
- Remove load immediately once goods are dry
- Maximum 100 washes

STORAGE

- It is recommended that the garments are stored in their original packaging clear of UV sources. Following use and cleaning, garments should be stored in a cool, dry and clean environment.

Components

- **Body:** Bakewell (Toray), 99.3% Polyester, 0.7% Carbon Graft
- **Cuff:** Polyester
- **Thumb Loop:** Braided Elastic
- **Tie:** Polyester Herringbone Weave
- **Binding:** Polyester Blend

Sizing

- **Item:** One size
- **Center Back Length (cm):** 120
- **Width (cm):** 160
- **Under Arm Length including cuff (cm):** 64.5

More information:

EUROPE

www.globusgroup.com

E: sales@globus.co.uk

T: +44 (0)161 877 4747

F: +44 (0)161 877 4746

MIDDLE EAST AND AFRICA

www.globusgroup.com/gcc

E: gcc@globusgroup.com

T: +971 4 882 9962

F: +971 4 882 9963

